**LIFE
LAB1**

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Tick-borne encephalitis IgG antibodies test shows:
 - whether the person is immune to tick-borne encephalitis after recovery or suspicion of asymptomatic tick-borne encephalitis;
 - whether sufficient antibodies have been developed after vaccination against the tick-borne encephalitis;
 - when revaccination is recommended: immediately, after 1, 2 or 3-5 years.
 The test is performed in a licensed laboratory by enzyme-linked immunosorbent assay (ELISA) from dried blood spot samples. The test kit is an approved in vitro diagnostic medical device for the collection of capillary blood samples at home. It is enough to apply 2 drops of blood to the sample card. The card with the samples is sent to the laboratory in a prepaid envelope. Test results are provided online at <https://lifelab1.com/testresults/>

1 234567 890128 >

LIFE LAB1

UAB „Laboratorija 1“
 Pamėnkalnio g. 36-2
 01114 Vilnius
 Lithuania

Tear at notch to open.

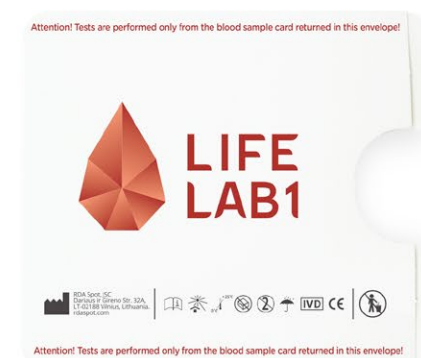
BLOOD SAMPLE RETURN BAG

IMPORTANT

DO NOT OPEN bag until you are ready to place card with samples into bag.

DO NOT REMOVE CONTENTS from bag.

Be sure bag is **COMPLETELY SEALED** before placing it in return envelope.

INSTRUCTION

Acceptable blood spots

- Wash hands with soap in warm water and dry them.
- Remove the card from the card's sleeve. Tear off the "SAVE" part of the card and keep it for your test results.
- Stimulate blood flow by waving your hand down for ~20 seconds.
- Wipe the middle finger with alcohol wipe and let it dry.
- Prick tip of the middle finger with lancet*. Remove the transparent safety cap of the lancet. Place the smaller end of the lancet against the side of the middle finger tip. Press the lancet firmly into the finger until you hear a click**. Do not share sterile lancets with other people. Always use a new sterile lancet. Lancets are for single use only. It is very important to safely discard the used lancet after each use in order to avoid accidental injury by lancet. Be sure to follow the requirements of local legislation for proper disposal.
- Drip one large blood drop in each circle. Squeeze the finger lightly and allow a large hanging blood drop to form. Be patient! Place the hanging droplet above the circle on the specimen area. Let the drop, but not the finger, come in contact with the paper. Repeat procedure placing single drops in all circles. If there is no blood flow, prick the second finger. Do not touch the specimen area of the card!
- Attach the band-aid. Write the date of sampling on the card.
- Place the card into the card's sleeve and let it dry for 3 hrs.
- Place the sleeve with the card into aluminium foil bag and close it tightly. Do NOT remove drying agent inside the aluminium foil bag! Place the aluminium foil bag into the paper envelope***. The sender's address should not be indicated on the outer envelope in order to preserve your confidentiality. Post it immediately, at the latest on the following day.
- Check your results at <https://lifelab1.com/testresults/> after 7-10 working days with your unique code, which is on the tear-off flap of the card. Test results are provided in standard units - VEU / ml.

Important! By sending your blood samples to the laboratory, you agree that Lifelab1 will analyse your health data and provide it to you on its website. If you later wish to remove your personal data from the database, please send us your unique card number via email info@lifelab1.com.

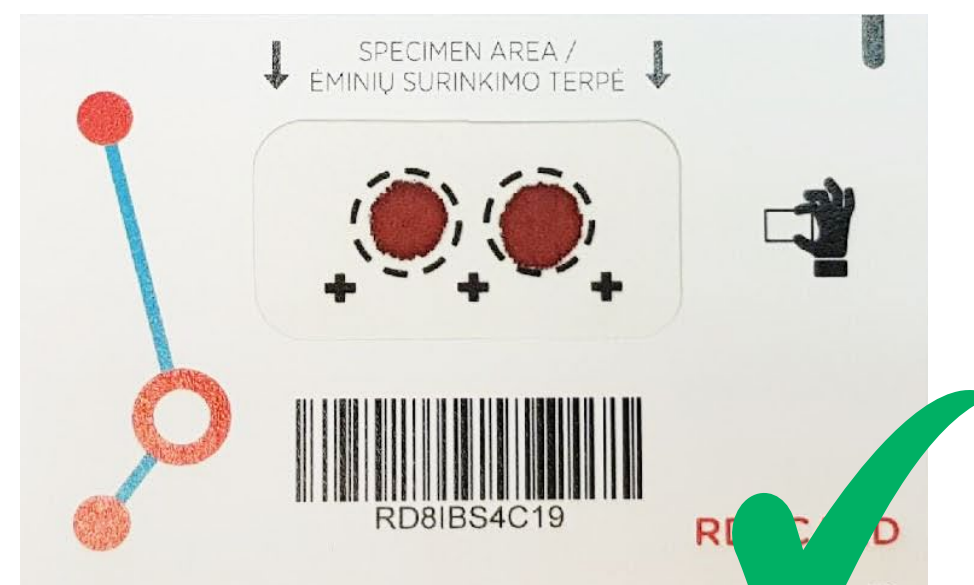
The use of this product must be in complete accordance with the enclosed directions. Lifelab1 assumes no liability for damages caused by the product being used in any other way. Keep out of reach of children!

The test is not intended to diagnose an acute infection. In case of suspicion, it is necessary to consult a doctor. Test results and recommendations indicate the expected level of risk, but do not replace the recommendations, examination, or treatment of a physician or other health care professional. Test results and recommendations must be viewed in context of the time in which they were given.

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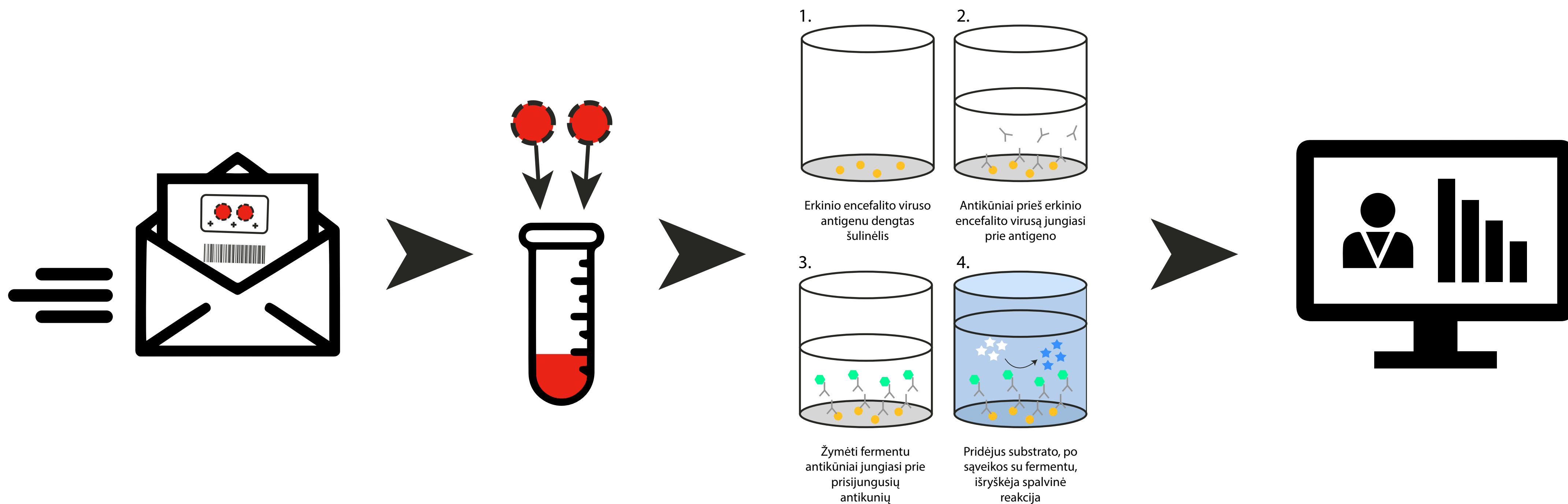
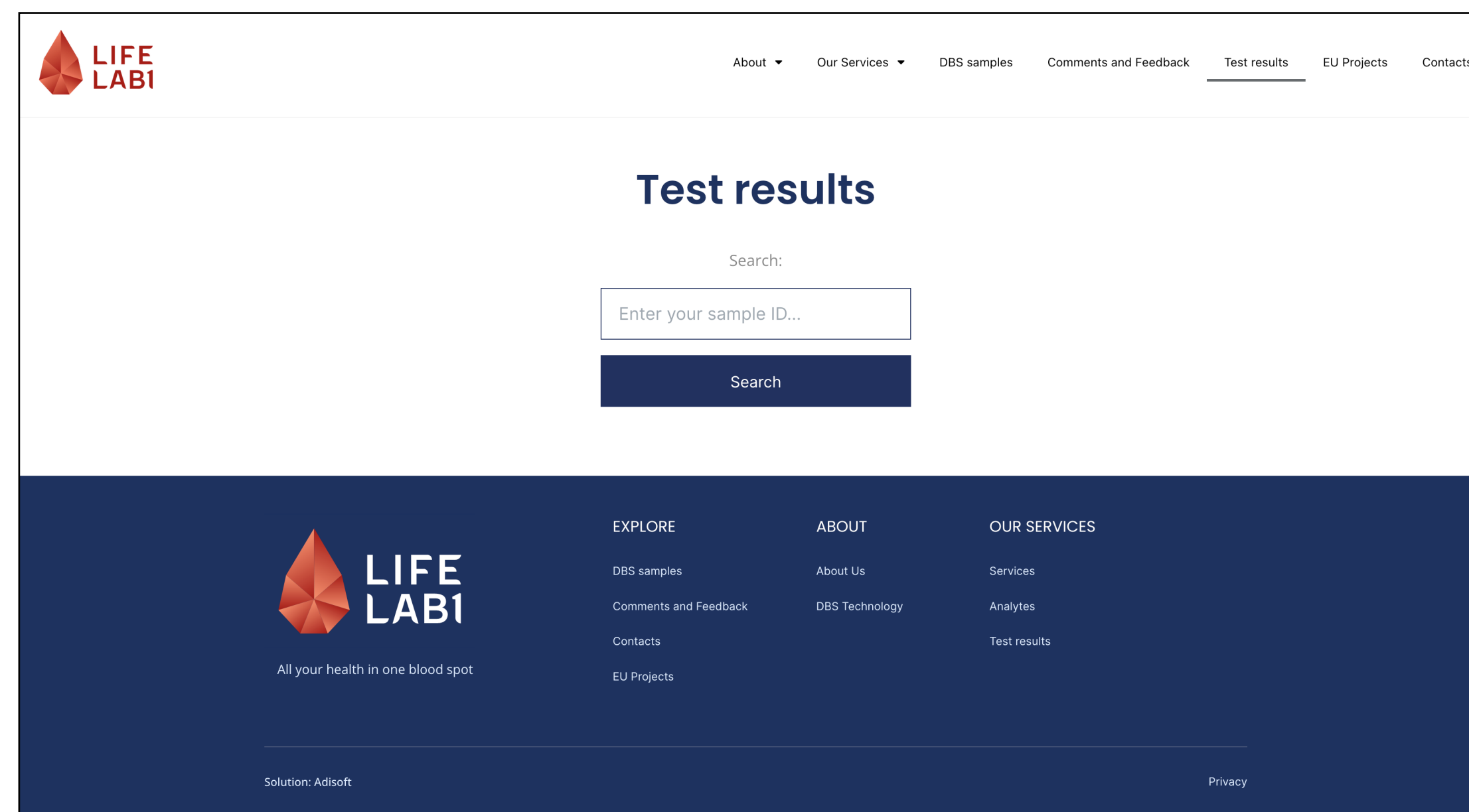
The screenshot shows the 'Test results' page on the LIFE LAB1 website. At the top left is the LIFE LAB1 logo. The top navigation bar includes links for 'About', 'Our Services', 'DBS samples', 'Comments and Feedback', 'Test results' (which is underlined), 'EU Projects', and 'Contacts'. The main heading is 'Test results'. Below it is a search section with the label 'Search:' and a text input field containing the placeholder 'Enter your sample ID...'. A dark blue 'Search' button is positioned below the input field. The footer is dark blue and contains the LIFE LAB1 logo with the tagline 'All your health in one blood spot'. It also features three columns of navigation links: 'EXPLORE' (DBS samples, Comments and Feedback, Contacts, EU Projects), 'ABOUT' (About Us, DBS Technology), and 'OUR SERVICES' (Services, Analytes, Test results). At the bottom left, it says 'Solution: Adisoft' and at the bottom right, 'Privacy'.

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